


Welcome to HeartBug!

Wear the Bug all the time and press "Make a Recording" when you feel unwell

This is the most important button!

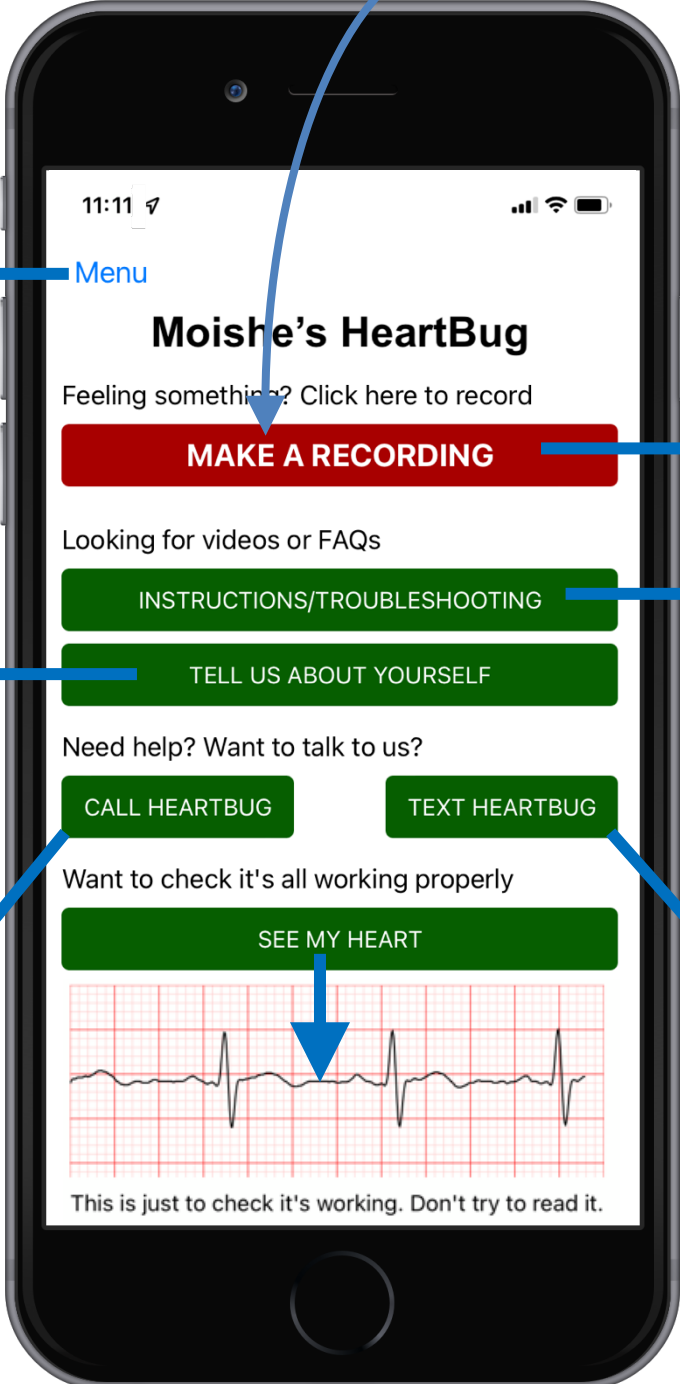

28 days left
HeartBug
How to put on
FAQs & troubleshooting
Swap HeartBug
More Stickers Please!

Menu

So we can look after you best, please tell us a bit about yourself:
What symptoms are you feeling?
Palpitations
How often are you feeling this?
Every day
Any known heart conditions or stroke?
No
Any previous heart procedures?
No
Taking any medications?
Blood pressure
SUBMIT

WHAT WERE YOU FEELING?
Palpitations
Fast heart rate
Chest pain
Short of breath
The "Big One"
Just Testing!
Something else
SUBMIT
Feeling something? Click here to record
RECORDING... (57s left)

Moishe's HeartBug
Feeling something? Click here to record
MAKE A RECORDING
Looking for videos or FAQs
INSTRUCTIONS/TROUBLESHOOTING
TELL US ABOUT YOURSELF
Need help? Want to talk to us?
CALL HEARTBUG
TEXT HEARTBUG
Want to check it's all working properly
SEE MY HEART
This is just to check it's working. Don't try to read it.

INSTRUCTIONS/TROUBLESHOOTING
TELL US ABOUT YOURSELF
CALL HEARTBUG
TEXT HEARTBUG
SEE MY HEART

Welcome to HeartBug!
Relax, this is going to be easy!
Here are the top 7 things to remember

Heartbug Office
calling main...
mute keypad speaker

Hi Moishe! Welcome to HeartBug. We're here to help, so if there's anything we can do, just let us know. The friendly HeartBug team.

Frequently Asked Questions

How to manage your stickers

These super stickers are designed to stay on for the whole monitoring period! However if they start falling off please call us and we will step you through the process of seeing if they need changing - as long as you can “see my heart” the stickers are working.

My skin is getting sore from the stickers, what can I do?

When replacing stickers, you can put the new sticker slightly to the side of the sore position, and put some cream on the sore part to heal it.

There's also a fabulous hydrocortisone spray called DermAid 1% which you can buy without prescription at pharmacies. It works wonders on the sore part, and also under the new sticker before you put it on, to prevent soreness.



How do I get more stickers?

Tap Menu | More Stickers Please! We will post them out to you the same day.

How much longer can I wear this awesome monitor?

Tap Menu, the number of days left is shown on the red part at the top.

Do I have to wear it if I'm not having symptoms/feeling sick?

YES – wear it the entire time. DON'T take it off.

The HeartBug is continually monitoring and will record if it sees something unusual. There may be things happening while you sleep, or that you don't feel. Your doctor will want to know about these.

Is the HeartBug waterproof?

YES – it is good enough for a shower; it can't be submerged completely, so please take it off for swimming or a bath.

Why would you tap “See my heart”?

To check that everything is working - if you see your heart beating, it's all working.

To see if the signal quality is good and the stickers are still OK. If it's really fuzzy, call us for advice.

NOT to diagnose yourself (unless you're a cardiologist). Sadly we can't discuss results with you either.